


ClearSight System Technology Overview

How does it work?


Volume clamp method

- The essence is to dynamically provide equal pressures on either side of the wall of the artery by clamping the artery to a certain constant volume
- 1000 times each second the cuff pressure is adjusted to keep the diameter of the finger arteries constant
- Continuous recording of the cuff pressure results in a real-time finger pressure waveform¹

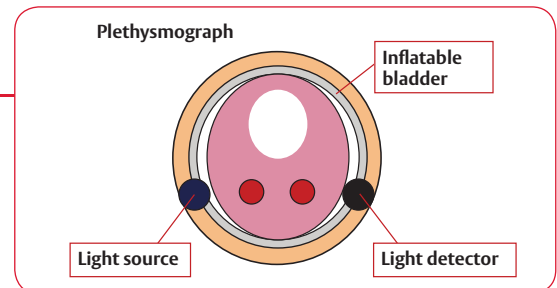


The Physiological method – Physiological Calibration

- The Physiological method is the real-time expert system that determines the proper arterial 'unloaded' volume, i.e. no pressure gradient across the arterial wall
- Automatic, periodic adjustments are essential to track the unloaded volume clamp setpoint when smooth muscle tone changes (e.g. during vasoconstriction)
- Calibration interval starts at 10 beats, but it increases to every 70 beats as stability increases
- Physiological interval >30 beats is considered reliable²

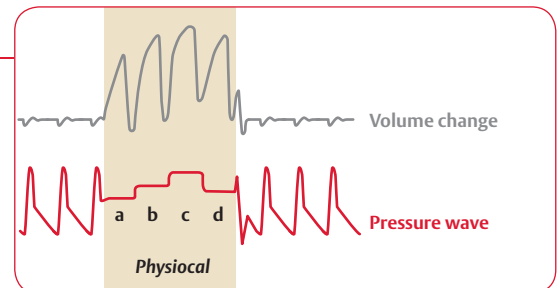


Brachial pressure reconstruction

- Clinical standard for noninvasive BP is brachial level
- The ClearSight system reconstructs the brachial arterial pressure waveform from the finger arterial pressure waveform
- The reconstruction algorithm is based on a vast clinical database³

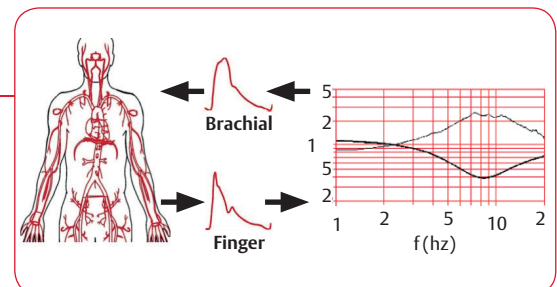

Cardiac output calculation

- Stroke volume is calculated by an algorithm based on an improved pulse contour method using:
 - The area under systolic portion of blood pressure curve (Systolic Pressure-time Integral - SPI)
 - A physiological model to calculate afterload individualized by age, gender, height and weight
- Cardiac output results from stroke volume times heart rate and is updated every beat⁴

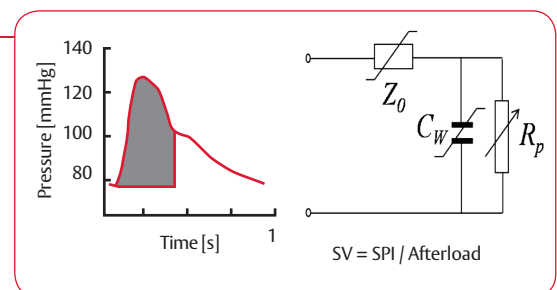

References

1. Peñáz J. Photoelectric measurement of blood pressure, volume and flow in the finger. 1973; Dresden 1973. p. 104
2. Wesseling KH, Wit B, Hoeven GMA, Goudoever J, Settels JJ. Physiological, calibrating finger vascular physiology for Finapres. Homeostasis. 1995;36:67–82.
3. Gizdulich P, Prentza A, Wesseling KH. Models of brachial to finger pulse wave distortion and pressure decrement. Cardiovasc Res. 1997;33:698–705. doi: 10.1016/S0008-6363(97)00003-5
4. Truijen J, van Lieshout JJ, Wesseling WA, Westerhof BE. Noninvasive continuous hemodynamic monitoring. J Clin Monit Comput. 2012 Jun 14.



Edwards

Validation of monitoring blood pressure and cardiac output with non-invasive finger cuff technology versus traditional methods

Blood pressure validation studies

BP can be measured reliably according to AAMI standard, bias <5 and SD < 8 mmHg

Vs. Noninvasive upper arm cuff		Bias ± SD
Akkermans et al. – Hypertension in Pregnancy 2009 ⁵	33 pregnant patients	SYS 2.3±6.8 / DIA 0.8±6.3
Eeftinck Schattenkerk et al. – Am J Hypertension 2009 ⁶	104 volunteers	SYS 4.3±9.3 / DIA -2.5±8.1
Vs. Invasive radial line		
Martina et al. – Anesthesiology 2012 ⁷	50 cardiac surgery patients	MAP 2.2±6.4 mmHg
Fischer et al. – Brit J Anaesthesia 2012 ⁸	44 cardiac surgery patients	MAP 4.6±6.5 mmHg
Martina et al. – ASAIO J 2010 ⁹	18 patients during CPB	MAP -1.3±6.5 mmHg
Vos et al. – Brit J Anaesthesia 2014 ¹⁰	112 OR patients	MAP 2.0±9.0 mmHg
Balzer et al. – J Inter Med Res 2016 ¹⁸	20 mod ortho patients	MAP -1±13 mmHg
De Wilde et al. – Anaesthesia 2016 ¹⁹	19 GI patients	MAP 3.5±5.2 mmHg
Heusdens et al. – Brit J Anaesthesia 2016 ²⁰	25 vascular patients	MAP 1.1±7.4 mmHg
Berkelmans et al. – J Clin Monit Comput 2017 ²¹	31 AF patients in ICU/MCU	MAP 0±8 mmHg
Rogge et al. – Anesthesia & analgesia 2018 ²²	35 obese patients	MAP 0.82±5.03 mmHg
Noto et al. – EU J of Anaesthesia 2018 ²³	30 awake vascular patients	MAP -6.8±6.7 mmHg

Cardiac output validation studies

Measurement of relative changes in CO performs comparably with invasive measurements

Vs. Pulmonary thermodilution		Percentage error
Bubenek-Turconi et al – Anesthesia-Analgesia 2013 ¹¹	28 cardiac surgery patients	38%
Bogert et al – Anaesthesia 2010 ¹²	25 post CABG patients	30%
Stover et al – BMC Anesthesiology 2009 ¹³	10 severely ill ICU patients	29%
Vs. Transpulmonary thermodilution		
Broch et al – Anaesthesia 2012 ¹⁴	40 cardiac surgery patients	23% / 26%
Hofhuizen et al. – J of Critical Care 2014 ¹⁵	20 post-cardiac patients	38.9%
Vs. Trans-thoracic echo-Doppler		
van der Spoel et al. – J Clin Anesth 2012 ¹⁶	40 ASA 1-2 patients	39%
Vs. Esophageal echo-Doppler		
Chen et al. J Clin Anesth 2012 ¹⁷	25 ASA 1-3 patients	37%

References (continued)

- Akkermans J, Diepeveen M, Ganzevoort W, van Montfrans GA, Westerhof BE, Wolf H. Continuous non-invasive blood pressure monitoring, a validation study of Nexfin in a pregnant population. *Hypertens Pregnancy*. 2009 May; 28(2):230-42. doi: 10.1080/10641950802601260
- Eeftinck Schattenkerk DW, Van Lieshout JJ, Van den Meiracker AH, Wesseling KR, Blanc S, Wieling W, Van Montfrans GA, Settels JJ, Wesseling KH, Westerhof BE. Nexfin noninvasive continuous blood pressure validated against Riva-Rocci/Korotkoff. *Am J Hypertens*. 2009;22:378-83.
- Martina JR, Westerhof BE, Van Goudoever J, de Beaumont EM, Truijen J, Kim YS, Immink RV, Jobsis DA, Hollmann MW, Lahpor JR, De Mol BA, Van Lieshout JJ. Noninvasive continuous arterial blood pressure monitoring with nexfin. *Anesthesiology*. 2012;116:1092-103.
- Fischer MO, Avram R, Cărlău I, Massetti M, Gérard JL, Hanouz JL, Fellahi JL. Non-invasive continuous arterial pressure and cardiac index monitoring with Nexfin after cardiac surgery. *Br J Anaesth*. 2012 Oct;109(4):514-21.
- Martina JR, Westerhof BE, Van Goudoever J, De JN, Van Lieshout JJ, Lahpor JR, De Mol BA. Noninvasive blood pressure measurement by the Nexfin monitor during reduced arterial pulsatility: a feasibility study. *ASAIO J*. 2010;56:221-7.
- Vos JJ, Poterman M, Mooyaert EA, Weening M, Struys MM, Scheeren TW, Kalmar AF. Comparison of continuous non-invasive finger arterial pressure monitoring with conventional intermittent automated arm arterial pressure measurement in patients under general anaesthesia. *Br J Anaesth*. 2014 Jul;113(1):67-74.
- Bubenek-Turconi SI, Craciun M, Miclea I, Perel A. Noninvasive Continuous Cardiac Output by the Nexfin Before and After Preload-Modifying Maneuvers: A Comparison with Intermittent Thermodilution Cardiac Output. *Anesth Analg*. 2013 Aug;117(2):366-72.
- Bogert LW, Wesseling KH, Schraa O, Van Lieshout EJ, De Mol BA, Van GJ, Westerhof BE, Van Lieshout JJ. Pulse contour cardiac output derived from non-invasive arterial pressure in cardiovascular disease. *Anaesthesia*. 2010;65:1119-25.
- Stover JF, Stocker R, Lenherr R, Neff TA, Cottini SR, Zoller B, Bécher M. Noninvasive cardiac output and blood pressure monitoring cannot replace an invasive monitoring system in critically ill patients. *BMC Anesthesiol*. 2009 Oct 12;9:6. doi: 10.1186/1471-2253-9-6.
- Broch O, Renner J, Gruenewald M, Meybohm P, Schottler J, Caliebe A, Steinfath M, Malbrain M, Bein B. A comparison of the Nexfin and transcardiopulmonary thermodilution to estimate cardiac output during coronary artery surgery. *Anaesthesia* 2012;67:377-83.
- Hofhuizen C, Lansdorp B, van der Hoeven JG, Scheffer GJ, Lemson J. Validation of noninvasive pulse contour cardiac output using finger arterial pressure in cardiac surgery patients requiring fluid therapy. *Journal of Critical Care* 2014 Feb;29(1):161-5.
- van der Spoel AG, Voogel AJ, Folkers A, Boer C, Bouwman RA. Comparison of noninvasive continuous arterial waveform analysis (Nexfin) with transthoracic Doppler echocardiography for monitoring of cardiac output. *J Clin Anesth*. 2012 Jun;24(4):304-9. doi: 10.1016/j.jclinane.2011.09.008.
- Chen G, Meng L, Alexander B, Tran NP, Kain ZN, Cannesson M. Comparison of noninvasive cardiac output measurements using the Nexfin monitoring device and the esophageal Doppler. *J Clin Anesth*. 2012 Jun;24(4):275-83. doi: 10.1016/j.jclinane.2011.08.014.
- Felix Balzer, Marit Habicher, Michael Sander, Julian Sterr, Stephanie Scholz, Aarne Feldheiser, Michael Müller, Carsten Perka and Sascha Treskatsch. Comparison of the noninvasive Nexfin monitor with conventional methods for the measurement of arterial blood pressure in moderate risk orthopedic surgery patients. *Journal of International Medical Research*, 44(4):832-43, 2016
- R.B.P. de Wilde, F. de Wit, B.F. Geerts, A.L. van Vliet, L.P.H.J. Aarts, J. Vuyk and J. R.C. Jansen. Non-invasive continuous arterial pressure and pulse pressure variation measured with Nexfin in patients following major upper abdominal surgery: a comparative study. *Anaesthesia* 2016, 71, 788-797
- J.F. Heusdens, S. Lof, C.W.A. Pennekamp, J.C. Specken-Welleweerd, G.J. de Borst, W.A. van Klei, L. van Wolfswinkel and R.V. Immink. Validation of non-invasive arterial pressure monitoring during carotid endarterectomy. *British Journal of Anaesthesia*, 117 (3): 316-23 (2016)
- G. F. N. Berkelmans, S. Kuipers, B. E. Westerhof, A. M. E. Spoelstra, de Man, Y. M. Smulders. Comparing volume-clamp method and intra-arterial blood pressure measurements in patients with atrial fibrillation admitted to the intensive or medium care unit. *J Clin Monit Comput* DOI 10.1007/s10877-017-0044-9
- Dorothea E. Rogge, Julia Y. Nicklas, Gerhard Schön, Oliver Grothe, Sebastian A. Haas, Daniel A. Reuter, and Bernd Saugel. Continuous Noninvasive Arterial Pressure Monitoring in Obese Patients During Bariatric Surgery: An Evaluation of the Vascular Unloading Technique (ClearSight system). 2019 Mar;128(3):477-483. doi: 10.1213/ANE.0000000000003943.
- Alberto Noto, Filippo Sanfilippo, Giovanni De Salvo, Claudia Crimi, Filippo Benedetto, Ximena Watson, Maurizio Cecconi and Antonio David. Noninvasive continuous arterial pressure monitoring with ClearSight during awake carotid endarterectomy. *Eur J Anaesthesiol* 2018; 35:1-9

For professional use. For a listing of indications, contraindications, precautions, warnings, and potential adverse events, please refer to the Instructions for Use (consult eifu.edwards.com where applicable).

Edwards devices placed on the European market meeting the essential requirements referred to in Article 3 of the Medical Device Directive 93/42/EEC bear the CE marking of conformity

Edwards, Edwards Lifesciences, the stylized E logo, ClearSight, Nexfin and Physiocal are trademarks of Edwards Lifesciences Corporation or its affiliates. All other trademarks are the property of their respective owners.

© 2021 Edwards Lifesciences Corporation. All rights reserved. PP-EU-1850 v2.0

Edwards Lifesciences • Route de l'Etraz 70, 1260 Nyon, Switzerland • edwards.com



Edwards